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EC REP MedPack Swiss GmbH, Switzerland, Tramstrasse 16, 9442 Berneck,

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POSTURE BRACE WITH STEEL STAYS

Indications for use:

Posture brace helps to limit volume of motions in lumbar and pectoral area of spine, ensures correct posture forming, abates pain in upper and lower spine parts. Prevents slouch and restores physiologically correct position of shoulder girdle. Is recommended to use during back pains caused by radiculitis, osteochondrosis, slipped and protruded disks, spinal curvatures.

- Two removable metallic stiffeners provide reliable support to pectoral and lumbar areas of spine. The stiffeners can be simulated, if it necessary, depending on the degree of fixation.
- Wide straps smoothly distribute pressure onto shoulders
- Additional straps ensure required level of loins compression.
- Breathable light fabrics of the bandage ensures maximal comfort during use.

Use

Before use please consult with your doctor. Required Support size is chosen based on the waist circumference shown in the chart above. Support is recommended to wear directly on underwear. Please apply while you are lying on the back on the flat rough or semi-rough surface. Individual treatment and terms are determined by the physician.

Recommended duration of the support bearing:




- Begin from 2 hours per a day, gradually increasing to 4 hours a day without a break or up to 6 hours a day with 2 hours breaks.
- Individual treatment and terms are determined by the physician.

Contraindications:

Skin local diseases, individual intolerance of the raw material components.

Dimensions	Sizes			Max deviation
	S/M	L/XL	XXL	
Chest circumference on the scapulas level, cm	64-89	90-116	117-130	± 1,0 cm

How to get it on:

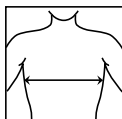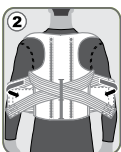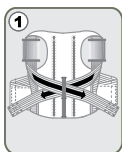Maintenance:

Wash with mild detergents at temperature no higher +35°C. Don't rub or wring out during washing and rinsing. Dry up away from sources of heat. Don't iron. Don't bleach. (Don't use chlorine-containing bleaching detergents). Don't use dry-cleaning.

Precautionary measures: Use the product exceptionally by prescription. Regulation and changing of the degree of support prop must be implemented only according to Your doctor's recommendations. Do not use the product during using of warming ointments.

Contents: polyether – 44%, latex – 21%, cotton – 15%, polyurethane foam – 15%, kapron – 5%, metallic splints

Storage conditions: store in dry place at temperature from +5°C to +40°C. Keep away from direct sun rays.

Shelf life: 36 months from manufacturing date observing storage conditions, exploitation and care of this product. After the termination of the storage period manufacturer does not guarantee sufficient elasticity of the product.

Manufacturing date: see on packaging.

Size: see on packaging.

EC REP MedPack Swiss GmbH, Switzerland, Tramstrasse 16, 9442 Berneck, phone: +41 71 747 11 95, fax: +41 71 747 11 91, e-mail: info@medpack-group.com, www.medpack-group.com.

Technomedica LLC., 24-A Berezinskaya str, Dnepropetrovsk, Ukraine 49130, phone/fax: +38 056 378 99 56, e-mail: info@med-textile.com, www.medtextile-swiss.com.