# Automatic blood pressure monitor

## Model Control

EN

### Automatic blood pressure monitor

Model Control

User's manual

RU

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## **DEAR GAMMA AUTOMATIC BLOOD PRESSURE MONITOR OWNER**

Thank you for choosing Automatic Blood Pressure Monitor **Gamma® TM** model Control. We are sure that having appraised worthily the high quality and reliability of this device you will become a regular user of the products of Trademark **Gamma®**.

This system is a fully automatic, digital blood-pressure measuring device for use on the upper arm, it enables very fast and reliable measurement of the systolic and diastolic blood-pressure as well as the pulse frequency by way of the oscillometric method of measuring. This monitor features Irregular Heartbeat Detection, as well as memory for the last measurement. Before starting to use this device please study the user's manual carefully. The user's manual offers all information you need to measure your blood pressure and pulse correctly. For all questions concerning the device please contact your local distributor or **Gamma®** service center in your country.

### **ATTENTION**

This blood pressure monitor is designed to carry out self-control over blood pressure but NOT to make self-diagnosis of hypertension/hypotension. Please DO NOT diagnose by yourselves basing on the measurement results obtained with the blood pressure monitor. Please DO NOT execute self-treatment of high/low blood pressure and DO NOT change the methods prescribed without consulting your doctor.

Type BF applied part.

Read the instructions carefully before using this device.

# TABLE OF CONTENTS

---


<b>IMPORTANT INFORMATION ON BLOOD PRESSURE AND ITS MEASUREMENT.....</b>	<b>6</b>
What Is Blood Pressure?.....	6
Which Values Are Normal?.....	7
<b>ADVANTAGES OF AUTOMATIC BLOOD PRESSURE</b>	
<b>MONITOR MODEL CONTROL.....</b>	<b>9</b>
Irregular Heartbeat Detection.....	9
Fuzzy Logic Measuring Technology.....	9
<b>GETTING READY FOR MEASUREMENT.....</b>	
Safety Precautions .....	10
Description of the Blood Pressure Monitor.....	11
Insert or Replace Batteries.....	12
Using a Mains Adaptor.(Option to purchase additional) .....	13
Cuff Connection.....	14
<b>MEASUREMENT PROCEDURE.....</b>	
Fitting the Cuff.....	14
Taking a Single Measurement .....	15
Reading Measurement Results .....	16
Discontinuing a Measurement.....	16
<b>MEMORY FUNCTION .....</b>	
<b>ERROR MESSAGES/TROUBLESHOOTING.....</b>	
<b>CARE AND MAINTENANCE .....</b>	
<b>SYMBOL INFORMATION .....</b>	
<b>TECHNICAL SPECIFICATIONS .....</b>	
<b>WARRANTY .....</b>	<b>20</b>

# **IMPORTANT INFORMATION ON BLOOD PRESSURE AND ITS MEASUREMENT**

---

## **What Is Blood Pressure?**

---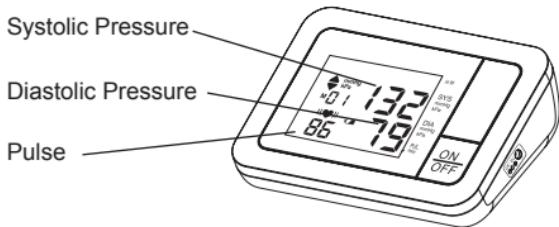

Blood pressure is the pressure that your blood extends to the vascular walls. Blood pressure is necessary to provide for constant blood flow inside the body. Thanks to it the cells get oxygen that provides for their normal functioning. The heart performs the function of a «pump», sending blood to the blood vessels. Each heart beat creates a certain level of the blood pressure.


There are 2 kinds of blood pressure: a systolic (upper) one, which corresponds to the heartbeat pushing blood into the arteriae; and a diastolic (lower) one, which means the blood pressure between two heartbeats.

Blood pressure is subject to fluctuations during the day even in healthy people. The fluctuations are influenced by a number of factors - time of day, person's condition, physical or mental activity, environment, etc.

### **Day-Night Fluctuations of the Blood Pressure**

An increase of blood pressure increases the burden onto the heart, affects blood vessels making their walls thick and less elastic.

One of the features of the hypertension is its ability to remain unnoticed for the patient at its early stages. That's why the self control of the blood pressure is so important. With the illness progressing, headaches and regular dizziness appear, the sight declines, the functioning of vitals (encephalon, heart, kidneys, blood vessels) breaks down. Without special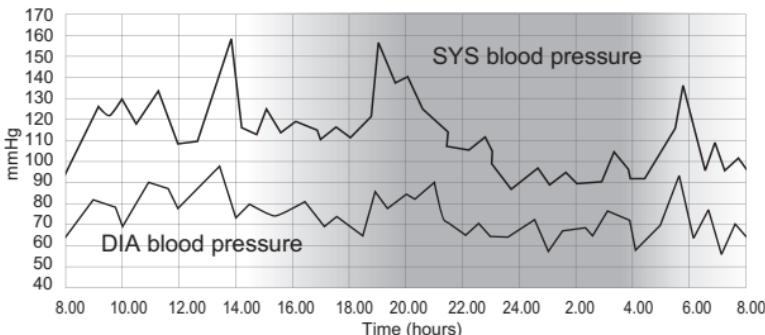

treatment the complications of hypertension might be kidney damages, breast-pang, paralytic stroke, aphasia, dementia, heart attack and stroke.

## Which Values are Normal?

Monitoring your routine blood pressure trend helps you to know your body condition. Human blood pressure naturally increases after reaching middle age.

This symptom is a result of continuous ageing of the blood vessels. Further causes include obesity, lack of exercise and cholesterol (LDL) adhering to the blood vessels, diabetes. Rising blood pressure accelerates hardening of the arteries, and the body becomes more susceptible to apoplexy and coronary infarction.

Definitions and Classification of blood pressure levels according to 2007 ESH-ESC Practice Guidelines for the Management of Arterial Hypertension: Data in mmHg.

Category	Systolic (mmHg)		Diastolic (mmHg)
Optimal	< 120	and	< 80
Normal*3	120-129	and/or	80-84
High normal	130-139	and/or	85-89
Grade 1 hypertension	140-159	and/or	90-99

Grade 2 hypertension	160-179	and/or	100-109
Grade 3 hypertension	$\geq 180$	and/or	$\geq 110$
Isolated systolic hypertension	$\geq 140$	and	< 90

Isolated systolic hypertension should be graded (1, 2, 3) according to systolic blood pressure values in the ranges indicated, provided that diastolic values are < 90mmHg.

Source: The European Society of Hypertension and European Society of Cardiology Task Force Members. 2007 ESH-ESC Practice Guidelines for the Management of Arterial Hypertension. *J Hypertens* 2007; 25:1751-1762.

- The diagnosis of hypertension requires from the patient to combine medical treatment prescribed by the doctor and mode of life correction.
- People with normal pressure and high normal pressure are recommended to carry out self-control of their tension in order to timely take measures to decrease the blood pressure level down to the optimal one without using any medications.
- For people more than 50 years old high level of systolic blood pressure (higher than 140 mmHg) is more crucial than diastolic pressure.
- Even with blood pressure being normal, people run the bigger risk of hypertension development with advancing age.

### **ATTENTION**

If you have normal results of blood pressure measured under calm conditions but your results are excessively high when measured under the conditions of physical or mental exhaustion, this might be a sign of so called brittle (that is unstable) hypertension. If you suspect that, please consult your doctor.

When measured correctly, if diastolic blood pressure is more than 120 mmHg, it is necessary to call the doctor immediately.

# ADVANTAGES OF AUTOMATIC BLOOD PRESSURE MONITOR MODEL CONTROL

---

## Irregular Heartbeat Detection

---

This function indicates allorhythmic heartbeating. If the symbol of IHD (⌚) appears on the display that means that certain abnormality in heart beat frequency was detected during the measurement.

In this case, the result may deviate from your normal blood pressure – repeat the measurement. In most cases, this is no cause for concern. However, if the symbol (⌚) appears on a regular basis (e.g. several times a week with measurements taken daily) we advise you to tell your doctor. Please show your doctor the following explanation:

### **Information for the doctor on frequent appearance of the Irregular Heartbeat Detector.**

This instrument is an oscillometric blood pressure monitor that also analyses pulse frequency during measurement. The instrument is clinically tested. The symbol (⌚) is displayed after the measurement, if pulse irregularities occur during measurement. If the symbol (⌚) appears more frequently (e.g. several times per week on measurements performed daily) we recommend the patient to seek medical advice. The instrument does not replace a cardiac examination, but serves to detect pulse irregularities at an early stage.

## Fuzzy Logic Measuring Technology

---

This unit uses the oscillometric method to detect your blood pressure. Before the cuff starts inflating, the device will establish a baseline cuff pressure equivalent to the air pressure. This unit will determine the appropriate inflation level based on pressure oscillations, followed by cuff deflation.

During the deflation, the device will detect the amplitude and slope of the pressure oscillations and thereby determine for you the systolic blood pressure, diastolic blood pressure, and pulse.

# **GETTING READY FOR MEASUREMENT**

---

## **Safety Precautions**

---

1. Use this device ONLY for the intended use described in this manual.
2. Do NOT use accessories which are not specified by the manufacturer.
3. Do NOT use the device if it is not working properly or damaged.
4. Do NOT use under any circumstances on newborns.
5. This device does NOT serve as a cure for any symptoms or diseases. The data measured are for reference only. Always consult your doctor to have the results interpreted.
6. Keep the equipment and its flexible cord away from hot surfaces.
7. Do NOT apply the cuff to areas other than the place directed.
8. Proper maintenance and periodically calibration are essential to the longevity of your device. If you are concerned about your accuracy of measurement, please contact local customer service for help.
9. This system is intended for individuals age 18 or above.
10. Do not use it for diagnosis of hypertension or testing on newborns, babies, young children or persons who cannot express their consent.
11. If you have been diagnosed with a severe arrhythmia or irregular heartbeat, atrial or ventricular premature beats or atrial fibrillation, measurements made with this instrument should only be evaluated after consultation with the doctor or your healthcare professionals.


**KEEP THIS MANUAL DURING THE WHOLE LIFETIME OF THE DEVICE**

## **Components of Your Blood Pressure Monitor Kit**

---


The blood pressure monitor kit includes Automatic Blood Pressure Monitor Model Control, M size cuff (22-32 cm), 4 batteries (AA), User's Manual, Warranty Card, Bag, Kit Box.

## Description of the Blood Pressure Monitor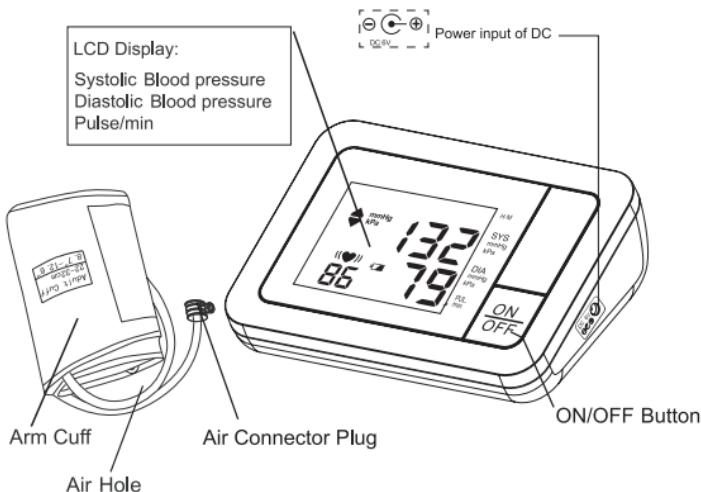

### Cuff:

Control offers a wide range cuff with D-ring, measuring size: 22~32cm, suitable for most of arm circumferences.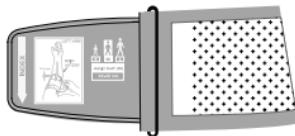

### Accessory:


Manual


Bag

## SYMBOLS ON DISPLAY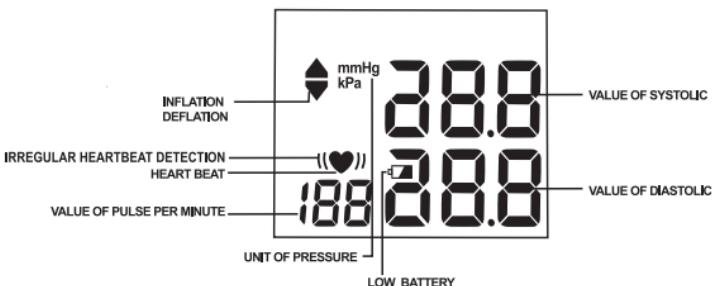

## Insert or Replace Batteries

1. Remove the battery cover.

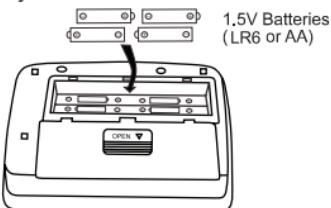2. Insert new batteries into the battery compartment as shown, taking care that the polarities (+) and (-) are correct.
3. Close the battery cover. Use only LR6, AA batteries.

Disposal of old batteries to the authorized collecting party subject to the regulation of each individual territory.

### CAUTION

- Insert the batteries as shown in the battery compartment.
- When (LOW BATTERY mark) blinks in the display, replace all batteries with new ones. Do not mix old and new batteries. It may shorten the battery life, or cause the device to malfunction.
- (LOW BATTERY mark) does not appear when the batteries run out.
- Battery life varies with the ambient temperature and may be shorter at

low temperatures.

- Remove the batteries if the device is not to be used for a long time.
- The batteries may leak and cause a malfunction.
- Use the specified batteries only. The batteries provided with the device are for testing monitor performance and may have a shorter life.

**NOTE:** To prolong the batteries' life the device switches off automatically if no button is pressed for 1 minute. Otherwise you can switch it off by pressing the START button.

## **Using a Mains Adaptor. (Option to purchase additional)**

---

It is possible to operate Automatic Blood Pressure Monitor Model Control with a mains adaptor (output 6 V DC/ 500 mA).

1. Connect the micro plug to the socket at the device right side.
2. Plug the mains adapter (output 6 V DC/ 500 mA) to a 100-240 V electrical power socket.


### **NOTE:**

- Please use the adaptor of Trademark **Gamma®**. Any other adaptor being used, the service center is not considered responsible for the warranty servicing/repair of the monitor.
- No power is taken from the batteries while the mains adaptor is connected to the device.
- Please consult service center if you have questions relating to the mains adaptor.

## Cuff Connection

---

Insert the cuff connector into the opening provided on the side of the device as shown in the picture.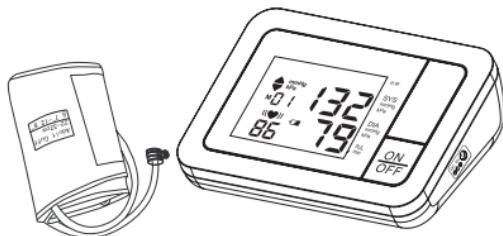

## MEASUREMENT PROCEDURE

---



### ATTENTION

- Find time to relax by sitting in a quiet atmosphere for some time before measurement.
- Efforts by the patient to support the arm can increase the blood pressure. Make sure you are in a comfortable, relaxed position and do not activate any muscles in the arm during measurement.
- Always measure on the same arm (normally left).
- Remove any garment that fits closely to your upper arm. Do not roll the sleeve since it can squeeze your hand and this can lead to false results.
- Use only clinically approved original cuff.
- If you want to follow the results of your blood pressure measurements, always perform measurements at the same time of day, since blood pressure changes during the course of the day.
- Measurements should be done after a 5 minute rest to ensure accuracy.

## Fitting the Cuff


---

- a. Plug the cuff connector to the cuff socket on the device left side.
- b. Assemble the cuff as shown below. The smooth surface should be inside of the cuff loop and the metal D-ring should not touch your skin.
- c. Stretch your left (right) arm in front of you with your palm facing up. Slide and place the cuff onto your arm to let the air tube and artery mark region toward the lower arm.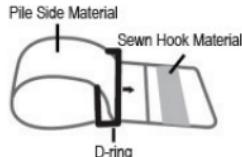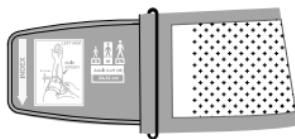

Wrap and tighten the cuff above your elbow. The artery mark on the edge of the cuff should be approx. 0.8" to 1.2" (2 cm to 3 cm) above your elbow. Align the tube over the main arteries on the inside.

- Leave a little free space between the arm and the cuff: you should be able to fit two fingers between them. Remove all clothing covering or constricting the measurement arm.

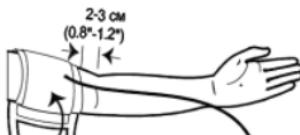- Press the hook material firmly against the pile material. The top and bottom edges of the cuff should be tightened evenly around your upper arm.

## Taking a Single Measurement

---

### ATTENTION

- Sit down for at least 10 minutes before measuring.
- Place your elbow on a flat surface. Relax your hand with the palm facing up.
- Make sure the cuff is about the same height as the location of your heart.

4. Remain still and do not talk or move during the measurement.

**WARNING:**

If the cuff is relatively lower (higher) than the heart, the obtained blood pressure value could be higher (lower) than the actual value.

**Always apply the pressure cuff before turning on the monitor.**

1. Press the ON/OFF button. The built-in microprocessor begins to inflate the cuff. In the display, the increasing cuff pressure is continually displayed.

2. After automatically reaching an individual pressure, the pump stops and the pressure slowly falls. The cuff pressure is displayed during the measurement. When the device has detected your pulse, the heart symbol on the display begins to blink and a beep tone is audible for every pulse beat.

## Reading Measurement Results

---

The measured systolic and diastolic blood pressure values, as well as the pulse are now displayed. The appearance of this symbol (♥) signifies that an irregular heartbeat was detected. This indicator is only a caution. It is important for you to be relaxed, remain still and do not talk during measurements.

**NOTE:** We recommend contacting your physician if you see this indicator frequently.

## Discontinuing a Measurement

---

If it is necessary to interrupt a blood pressure measurement for any reason (e.g. the patient feels unwell), the ON/OFF button be pressed at any time. The device then immediately lowers the cuff pressure automatically.

## MEMORY FUNCTION

---

Automatic Blood Pressure Monitor **Gamma®** TM Control automatically stores the result measured. To see the result of the last measurement press and hold the ON/OFF button for at least 3 seconds. If an irregular heartbeat interval was detected in the measurement stored in memory, the icon (♥) will be displayed.

## TROUBLESHOOTING

---

If you have trouble in using the unit please check the following points first.

ERROR DISPLAY	POSSIBLE CAUSE	HOW TO CORRECT
Nothing is displayed. When you push the POWER button or  battery icon flash	No battery installation  Battery worn out  The polarities of batteries placed wrongly	Insert batteries  Replace with new batteries  Insert battery in the correct polarities
E0	Testing pressure with Clothes  The marker of arm cuff  is not in alignment with the arm's artery.	Take off the clothes and test it again  Align the marker of arm cuff with the arm's artery, and measure again.
E1: Can't normally increase pressure	Check your wrist cuff for air leakage	Replace wrist cuff with new one
E3: Inflate pressure too high		Remeasure or send back to dealer to recalibrate
E2 E4: Hand shaking while measurement	Hand or body shaking while measurement	Keep static position and measure again
Battery icon on	Battery low power	Replace battery and measure again

The systolic pressure Value or diastolic Pressure value too high	1.The wrist cuff was held lower than your heart 2.The wrist cuff was not attached properly 3.You moved your body or spoke during measurement	Keep correct position and measure again
The systolic pressure value or diastolic pressure value too low	1.The wrist cuff was held higher than your heart 2.You moved your body or spoke during measurement	

## CARE AND MAINTENANCE

---

To avoid the monitor attracting dirt, dust or other contaminants, wash and dry your hands thoroughly before use.

### Cleaning

- To clean the monitor exterior, wipe it with a cloth moistened with tap water or a mild cleaning agent, then dry the device with a soft dry cloth. Do NOT flush with water.
- Do NOT use organic solvents to clean the monitor.
- Do NOT wash the pressure cuff.
- Do NOT iron the pressure cuff.

### Monitor Storage

- Storage condition: -20°C to 55°C (-4°F to 131°F), below 95% relative humidity.
- Always store or transport the monitor in its original storage case.
- Avoid dropping or heavy impact.
- Avoid direct sunlight and high humidity.

## SYMBOL INFORMATION

---

SYMBOL	REFERENT
	Consult instructions for use
	Manufacturer
	Serial number
	Caution, consult accompanying documents
	Dispose of in accordance with the requirements of your country
	Type BF Equipment
	Class II
	CE mark 0413

## SPECIFICATIONS

---

Measuring Method	Oscillometric Measurement
Indication	Digital LCD display
Measuring Range:	Pressure: (30~280) mmHg Pulse: (40~199) Beat/min
Accuracy:	Static Pressure: $\pm 3$ mmHg Pulse: $\pm 5\%$
Memory:	Memory for the last measurement
Power supply:	4x1.5V Batteries (LR6 or AA)

Operating condition:	+5°C~+40°C. 30% RH~80% RH Atmospheric pressure: 80 kPa~106 kPa
Storage condition:	-20°C~+55°C. 10% RH~93% RH Atmospheric pressure: 50 kPa~106 kPa
Dimensions:	Approx: 138(W) X 96(H) X 50(D) mm
Weight:	Approx: 355 g, excluding batteries
Classification	Type BF
Cuff Size	22-32 cm

\* Specifications may be changed without notice in the event of improvement being made.

This device has been tested to meet the electrical and safety requirements of:

IEC/EN 60601-1,  
IEC/EN 60601-1-2

Reference to Standards:

- EN 1060-1/-3, NIBP-requirements
- IEC60601-1 General requirement for safety
- IEC60601-1-2 Requirements for EMC
- EN1060-4, NIBP clinical investigation
- AAMI/ANSI/IEC 80601-2-30, ANSI/AAMI/ISO 81060-2, NIBP requirements

## **WARRANTY**

---

Your Automatic Blood Pressure Monitor Model Control is warranted for 5 years from date of purchase. Warranty for the cuff is 1 year from the date of purchase. The warranty does not apply to damage caused by improper handling, accidents, not following the operating instructions or self-maintained alterations made to the device. The warranty is only valid upon presentation of the warranty card which was correctly filled in and sealed.

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Lotus Global Co.,Ltd
■ Shenzhen Pango Electronic Co., Ltd No 25, 1st Industry zone,