Compressor Nebulizer System

Model Effect New

EN

Compressor Nebulizer System

Model Effect New

User's Manual

RU

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DEAR Gamma COMPRESSOR NEBULIZER OWNER

Thank you for choosing **Gamma®** Compressor Nebulizer model Effect New. We are sure that having appraised worthily the high quality and reliability of this device you will become a regular user of the products of trademark **Gamma®**.

Your nebulizer is intended for use in treatment of asthma, COPD and other respiratory ailments in which an aerosolized medication is required during therapy. Aerosol easily and quickly comes to both upper and lower respiratory tracks.

Before starting to use the compressor nebulizer please study the user's manual carefully. Following the rules stated in the user's manual will provide for the faultless operation of the compressor nebulizer for a long period of time. For all questions concerning the unit please contact service centre of distributor of trademark **Gamma®** in your country.

TABLE OF CONTENTS

INTRODUCTION.....	6
What is a nebulizer and how does it work?	6
What is nebulizing therapy used for?	6
What are the advantages of nebulizing therapy?	6
BEFORE STARTING TO USE THE UNIT	7
Important Safety Information	7
Unit Identification.....	8
Assembling the Unit.....	9
CARRYING OUT NEBULIZING THERAPY.....	10
Operating the Unit	10
Changing the Air Filter	10
UNIT MAINTENANCE.....	11
Cleaning and Disinfecting.....	11
Unit Storage.....	12
Troubleshooting	12
TECHNICAL SPECIFICATIONS	12
SYMBOL DESCRIPTION.....	14
WARRANTY.....	14

INTRODUCTION

What is a nebulizer and how does it work?

The compressor nebulizer system is a device that turns a liquid drug into a fine aerosol under the influence of the compressed air. Aerosol contains large and small particles. When breathing the gravity makes the larger particles accumulate on the walls of the upper respiratory passages while the smaller particles have enough time to reach the lower bronchial branches. Therefore the drug comes to all parts of the patient's bronchial tree, including the smallest bronchi and alveoli, and gets into the blood vessels on the walls of the alveoli in big therapeutic doses with tiny or none side effects.

What is nebulizing therapy used for?

Nebulizer systems are used by the patients with:

- acute respiratory diseases,
- obstructive pulmonary disease,
- chronic inflammatory processes in respiratory passages such as chronic bronchitis, bronchial asthma (incl. chronic or unstable ones, exacerbation of asthma), chronic pharyngitis,
- respiratory passages infections,
- abnormalities in secretion quality and secreting in bronchi,
- different types of coughing.

What are the advantages of nebulizing therapy?

Nebulizing therapy gives a possibility to quickly effect the inflammatory zone by big drug doses which increases the influence of the treatment and reduces the risk of side effects to other organs. Nebulizing therapy is the easiest way of inhalation treatment and it is also an economical means of treatments from the point of view of purposeful use of drug. Aerosol treatment with the help of nebulizing systems can be used for children, elderly and infirm patients.

BEFORE STARTING TO USE THE UNIT

Important Safety Information

- To avoid electrical shock: keep unit away from water.
- Do not touch the unit, plug or unplug into the electrical outlet with wet hands.
- Do not use the unit if it has any damaged parts (including plug), if it has been submersed in water or dropped. Promptly send the unit for examination and repair.
- The unit should not be used where flammable gas, oxygen or aerosol spray products are being used.
- Keep the air vents open. Do not place the unit on a soft surface where the openings can be blocked.
- If the medication cup is empty, do not attempt to operate the unit.
- Always keep the unit unplugged while not in use.
- Do not tilt or shake the unit when in operation.
- Disconnect the unit from the electrical outlet before cleaning, filling and after each use.
- Do not use attachments unless recommended by the manufacturer.
- Close adult supervision is highly recommended when the unit is used by children or invalids.
- Keep your eyes away from the output of medication mist.
- The maximum capacity of the medication cup is 8 ml and should not be overfilled.
- If any discomfort or abnormality occurs, stop using the unit immediately.
- Do not use the device if the air tube is bent.

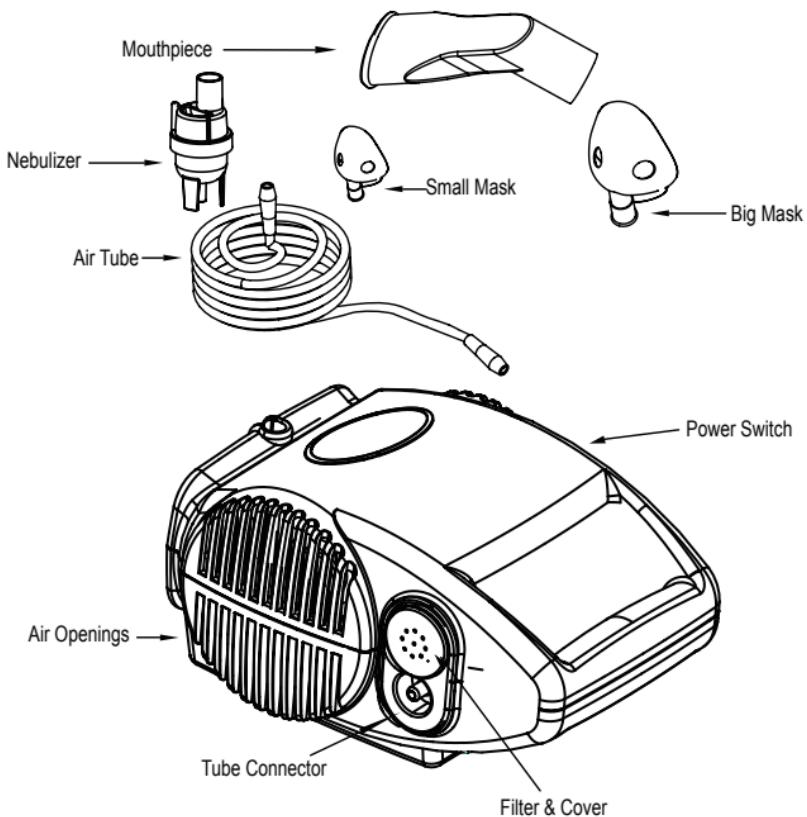
Electric devices must be disposed of in accordance with the locally applicable regulations, not with domestic waste.


KEEP THIS INSTRUCTION MANUAL DURING THE WHOLE SERVICE LIFE OF THE UNIT

Please use only those drugs that were prescribed to you by your doctor, DO NOT execute self-treatment and DO NOT change the methods/frequency prescribed without consulting your doctor.

Unit Identification

Assembling the Unit

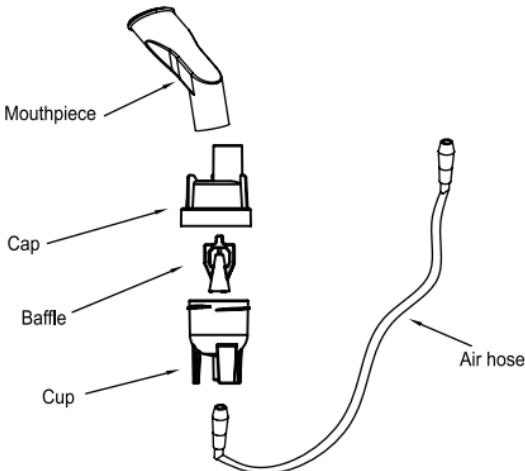Follow the cleaning instructions in this manual under «Cleaning the Unit» prior to using your nebulizer for the first time or after it has been stored for an extended period of time.

1. Place the compressor on a flat, stable surface within reach.
2. Gently twist and pull straight up on the lid of the nebulizer to separate into two parts (medication cup and cover).
3. Make sure that the nozzle is properly installed inside the medication cup. The stem inside the medication cup inserts into the tube of the nozzle.
4. Add the prescribed amount of medication to the medication cup.
5. Reassemble the nebulizer by carefully twisting the medication cup and cover together. Make sure that the two parts fit securely.

REMEMBER: Always unplug the compressor and make sure the power-switch is turned to the «OFF» position before cleaning, assembling and before or after each use.

CARRYING OUT NEBULIZING THERAPY

Operating the Unit

1. Attach one end of the air tube connector to the air output.
2. Carefully attach the opposite end of the air tubing connector to the stem at the base of the nebulizer kit.
3. Open a nebulizer and put inside the prescribed medicine but not more than 8 ml. Then close the nebulizer cover.
4. Attach the oral mouthpiece or mask to the top of the dispenser.
5. Connect the device to the main power. Your inhaler is ready for use.

After every use:

1. Unplug the appliance.
2. Allow the unit to completely cool.
3. Carefully detach the air tubing from the nebulizer and pour out any remaining medication.
4. Follow the cleaning procedures provided in this manual.

Changing the Air Filter

It is important to change the air filter approximately when the air filter turns gray.

1. Remove the air filter cover by gently pulling forward.
2. Discard the gray filter.
3. Replace with a new, clean air filter.
4. Securely re-attach the air filter cover to the unit.

NOTE: Air filters cannot be cleaned or washed. Only Effect New air filters can be used. Do not substitute alternate material such as cotton. Do not operate without an air filter.

UNIT MAINTENANCE

Cleaning and Disinfecting

- Check air filter, nebulizer, mouthpiece and any other optional components before each use. Dirty or worn parts should be replaced.
- Do not store the air tube with moisture or medication remaining in the air tube. This could result in infection as a result of bacteria.

Rinsing technique (performed after each treatment or before first use).

1. Make sure that the power-switch has been turned to the «OFF» position and the unit has been disconnected from the power source.
2. Disconnect the air tube from the nebulizer device.
3. Disassembly components of the nebulizer kit:
 - a take out the mouthpiece
 - b gently twist and pull up the cover of the nebulizer kit to open and separate
 - c disconnect the nozzle
4. Rinse the nebulizer kit and components with hot tap water.
5. Dry with clean towels or completely air dry.
6. Reassemble the nebulizer kit.

NOTE: For the first time cleaning or after the unit has been stored for an extended period of time, thoroughly clean all components, including the air tube. The nebulizer kit is dishwasher safe.

Cleaning the compressor.

Wipe the compressor daily using a soft cloth.

NOTE: Any other form of cleaning or cleaning agents may damage the finish of the unit.

Disinfecting

All nebuliser components (except air hose) can be disinfected with chemical disinfectants following dosage and use limitations as provided by disinfectant manufacturer.

Disinfectants are usually available at pharmacies.

Unit Storage

- Do not store the unit in direct sunlight, high temperature or humidity.
- Keep the unit out of reach of small children.

Troubleshooting

PROBLEM	ACTION
Unit does not operate when power switch is pressed.	Check the AC connection to the outlet.
No misting or low rate of misting.	<ul style="list-style-type: none">• Check that there is medication in the nebulizer cup.• Check the main unit if there is any physical damage.• Check the position of the nozzle inside the nebulizer.• Make sure that air tube and other components are properly attached.• Check the air filter and replace if necessary.

TECHNICAL SPECIFICATIONS

Power:	230 VAC, 50 Hz
Power Consumption:	< 65 W

Nebulizer Flow:	6 L
Compressor Flow:	8-10 L
Medication Capacity:	8 ml
MMAD:	2.44 um
Average nebulisation:	0.4 ml/min
Remain Amount:	0.15 ml
Noise Level:	below 55 dBA
Compressor Pressure Range:	30 to 36 Psi / 280 to 350 Kpa / 2.8~3.5bar
Operation Pressure Range:	8 to 16 Psi / 100 to 150 Kpa / 1~1.5bar
Operating Temperature Range:	10°C to 40°C (50°F to 104°F)
Operation Humidity:	10% to 95% RH
Storage Temperature Range:	-20°C to 70°C (-4°F to 158°F)
Storage Humidity Range	10% to 95% RH
Dimension (L x W x H):	205*155*113 mm
Products Weight:	1.55 Kg
Standard Accessories:	Compressor Nebulizer Machine Adult Mask Pediatric Mask Filter 5 pcs Air Tube Medical Cup Mouth Piece

SYMBOL DESCRIPTION

Protection against electric shock:

- Class II equipment.

Type BF applied parts:

- Mouthpiece and mask.

Protection against harmful ingress of water and particulate matter:

- IP21

Degree of safety in the presence of flammable anesthetics or oxygen:

- No AP/APG (not suitable for use in the presence of flammable anesthetics or oxygen).

WARRANTY

This device is guaranteed for 2 years from the date of purchase. The warranty does not cover the accessories supplied with the device and those parts subject to normal wear and tear. The guarantee is valid only on presentation of the guarantee card completed by the dealer confirming date of purchase and the receipt.

- Should the unit be damaged due to user misuse or negligence, the manufacturer or authorized dealers shall not be responsible for any damage or loss to the unit.
- All assembly, extension, adjustment or repair of this unit must be performed only by the authorized service center of **Gamma®** distributor in your country.
- For repairs or to purchase parts, please contact the authorized service center of **Gamma®** distributor in your country.

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